

Velocity PS Anterior Closure


5646PS

Warnings and Instructions: Review carefully, proper application is required


⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.

This device should not be worn by persons with known allergies to neoprene.


Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

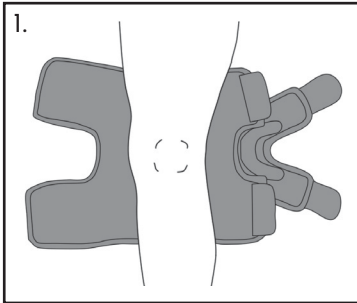


1. Loosen all straps and center brace on back of leg. Remove buttress from brace.

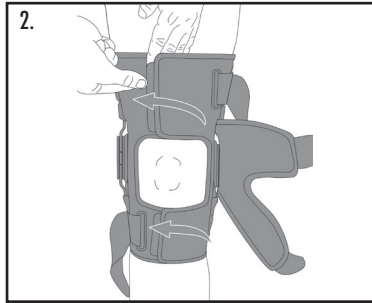


2. Pull upper and lower flaps and fasten

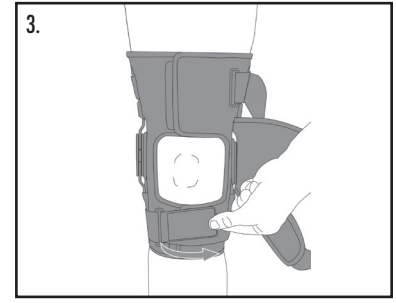


3. Pull lower leg strap through D-ring and secure.

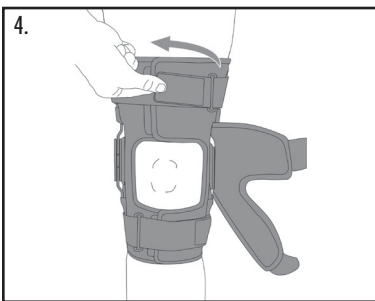


4. Pull upper leg strap through D-ring and secure.

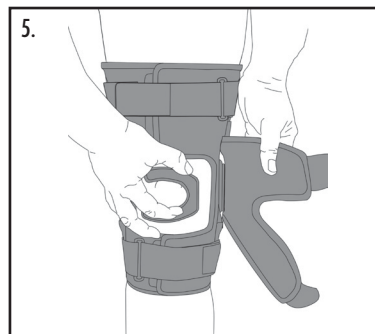


5. Place patella buttress on patella where support is needed. Hold buttress in place.

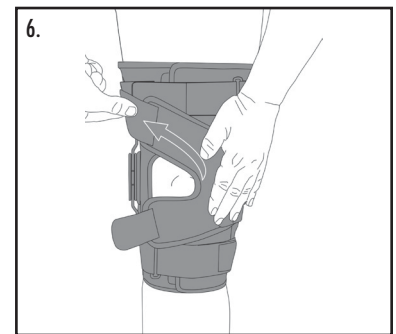

6. Maintain position of buttress while Pulling and fastening upper patella flap strap.

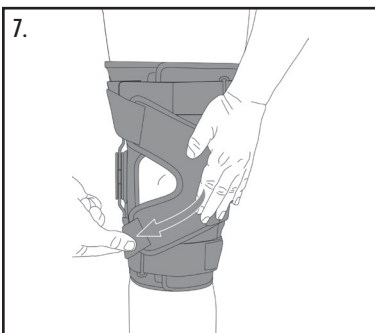


7. Pull and fasten lower patella flap strap.

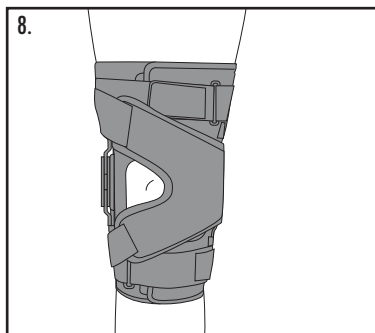


8. Completed application.



OPTIONAL: Reversing patella stabilizing flap to opposite side

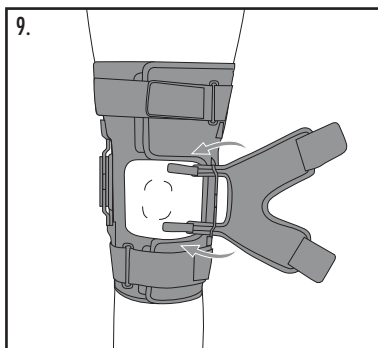

9. Detach flap hook from around hinge strut, and remove flap from brace.

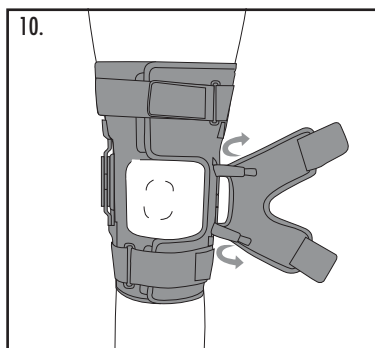

10. Re-attach flap by wrapping hook closures around hinge struts on opposite side. Pull flap snug against hinge. Attach hook and loop closures near edges of flap.

See Instructions on How to Adjust the Hinges Range of Motion on Back


HELIX & WEBER

© 2017 Weber Orthopedics Inc.
DBA Hely & Weber all rights reserved




Rev E 8/25


Adjusting the Range of Motion Hinges

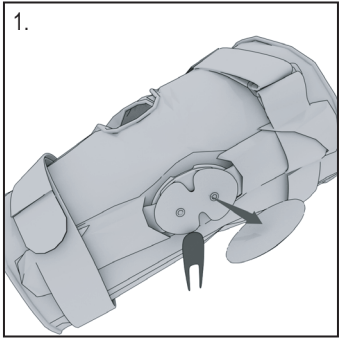


1. To remove the hinge cover cap, insert the provided tool into the space in the back of the hinge. The cap will come off using a twist or pry motion.

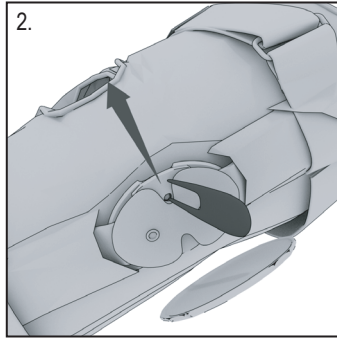


2. Remove the E10 stops that come pre-installed.

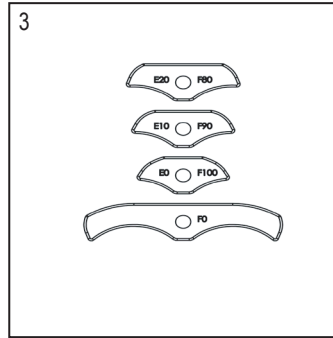

3. Choose the hinge stops with the desired degrees of extension and flexion.

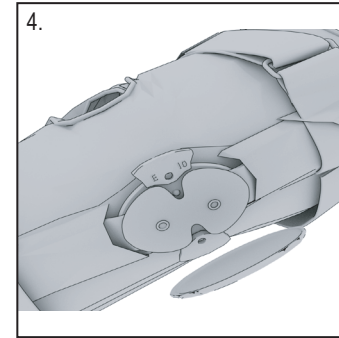

4. Install the extension stop (E) on the extension stop side of the hinge, and the flexion stop (F) on the flexion stop side of the hinge. The stops for both hinges must be set to the same settings.

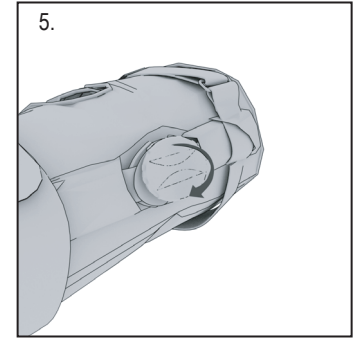

5. Apply the front edge of the cap to the hinge plate first, then press the posterior side to secure.

