


Universal Matt Strap

Warnings and Instructions: Review carefully, proper application is required


⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.

Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.


Sizing Customization

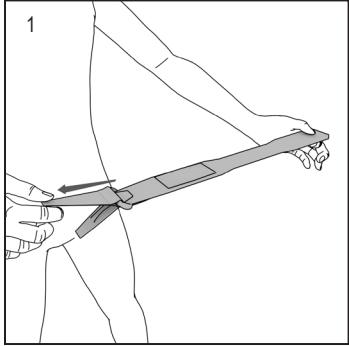


1. Detach both tabs.
Pull tab with ring to its full extension.

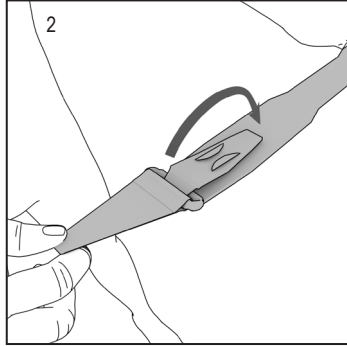


2. Fold back tab with logo and fasten.

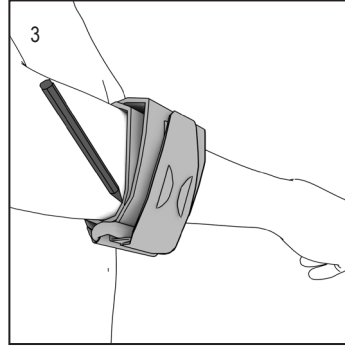

3. Fasten strap around affected area.
Mark on logo side of ring for trimming.

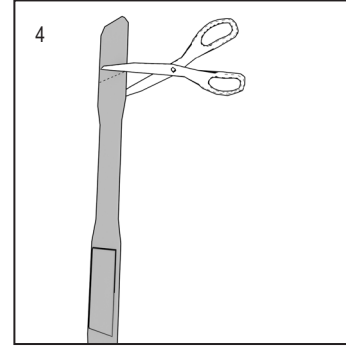


4. Remove brace and trim.

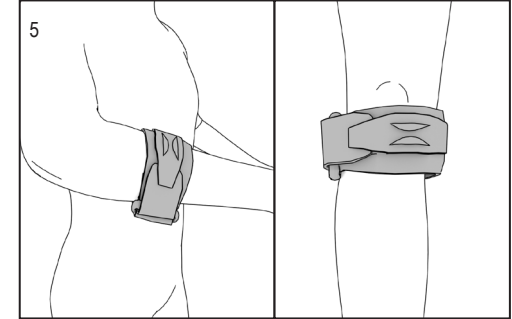


5. Finished application.


Leg Application

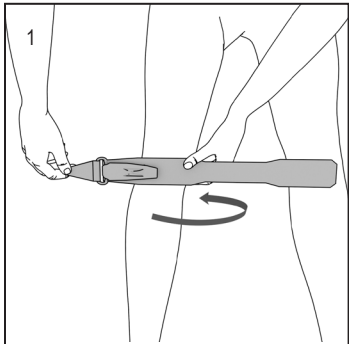


1. Place the moon shaped buttress over the affected area.
Wrap long end of strap around leg.

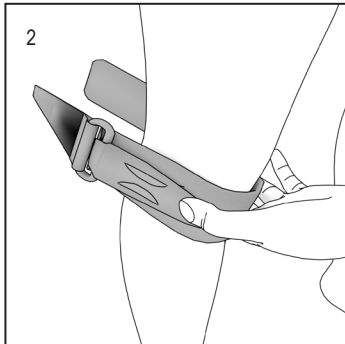


2. Mold aluminum buttress to affected area.

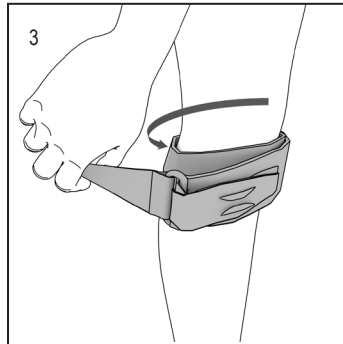

3. Hold in place and fasten strap.

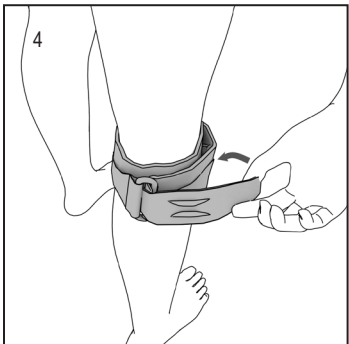


4. Detach, pull, and re-attach tab with logo.

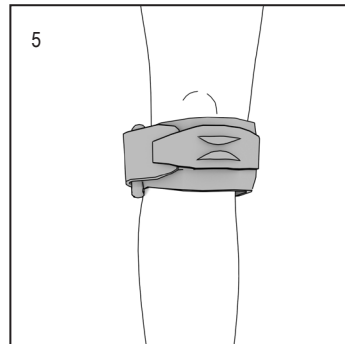


5. Finished application leg.




Arm Application

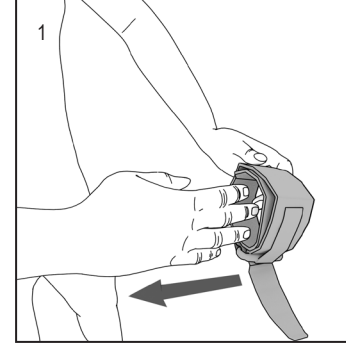


1. Adjust brace to size of affected area and fasten.
Detach tab with logo.
Slide brace on arm.
Place buttress over affected area.

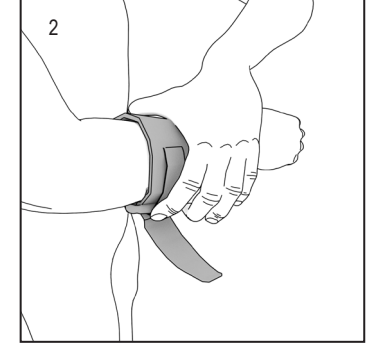

2. Mold aluminum buttress to affected area.

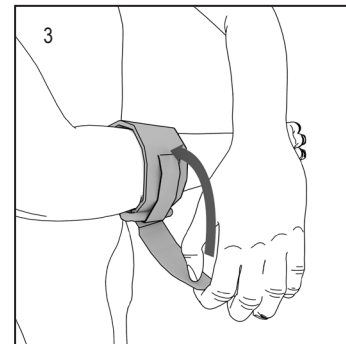

3. Pull and fasten tab with logo.

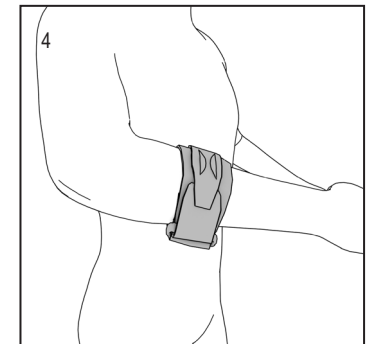

4. Finished application arm.