



7686 KNAPP™ II-12"
7688 KNAPP™ II-16"
HINGED KNEE ORTHOSIS

ORDERING
INFO

SIZING

MEASUREMENTS
ARE TAKEN
AT 6" ABOVE
MID-PATELLA

XS	13" - 15"	33 - 38 CM
S	15" - 18"	38 - 46 CM
M	18" - 20"	46 - 50 CM
L	20" - 23"	50 - 58 CM
XL	23" - 25"	58 - 64 CM
XXL	25" - 28"	64 - 71 CM

LARGER AND CUSTOM SIZES
FOR 7686/7688 MODELS
AVAILABLE UPON REQUEST

GREATER ROM LIMITS
AVAILABLE IN PROFESSIONAL
STOP KIT #X1224-K



DEVELOPED IN COLLABORATION WITH

THOMAS P. KNAPP, M.D.
Cedars-Sinai Kerlan-Jobe Institute



CALL TODAY! (800) 654-3241
EMAIL: ORDER@HELY-WEBER.COM



© 2019 HELY & WEBER



THE **ULTIMATE** IN LIGHT WEIGHT
LIGAMENT SUPPORT BRACING

FEATURES & BENEFITS

ANTERIOR CLOSURE

DESIGN ALLOWS THE BRACE
TO BE CONTOURED TO GIVE
A CUSTOM FIT

EASY TO ADJUST POP STOP
HINGE ALLOWS ADJUSTMENT
IN SECONDS

COMBINATIONS OF SPACER

MESH AND DOUBLE KNIT
MATERIAL PROVIDES
COMPRESSION SUPPORT
AND EDEMA CONTROL

INDICATIONS

- MENISCUS INJURIES
- STRAINS AND SPRAINS
OF THE KNEE





MAXIMIZING JOINT SUPPORT AND PATIENT COMFORT

TWO LENGTHS:

12" PERFECT FOR THE MCL,
LCL AND MENISCAL
INJURIES

16" THE ADDED LENGTH
AND STRAPS PROVIDE
GREATER SUPPORT TO
THE ACL, PCL

ANTERIOR CLOSURE

ALLOWS THE BRACE
TO BE CONTOURED
TO THE INDIVIDUAL'S
LEG, PROVIDING
A CUSTOM FIT

NEW POP STOP™ ROM HINGE ADJUSTS IN SECONDS (PATENT PENDING)


SLIP ROUNDED
PART OF TOOL
(PROVIDED)
UNDER FLEXION
SIDE OF CAP



REMOVE
CAP WITH
A GENTLY
TWISTING
MOTION



POP OUT
EXISTING
STOP WITH
FORKED END
OF TOOL
WHILE
PRESSING
DOWN POST
SLIDE STOP
OUT

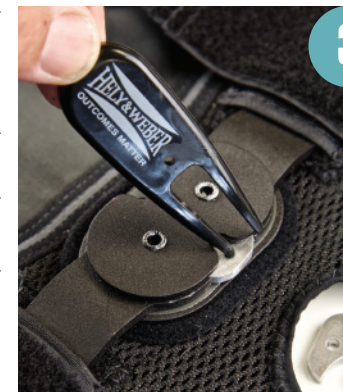

SELECT
DESIRED
STOP(S) AND
POP STOP
INTO PLACE
REPLACE CAP



NOTE: FULL
INSTRUCTIONS
FURNISHED
WITH BRACE



LOW PROFILE HINGE

SLEEK DESIGN

- KEEPS HINGE CLOSE TO THE KNEE
FOR OPTIMUM SUPPORT

POP STOP™ ROM SYSTEM (PATENT PENDING)

- ALLOWS SETTING CHANGES
IN SECONDS
- ADDITIONAL ROM STOPS INCLUDED
WITH BRACE

CONDYLE PADS

- FOR ADDED COMFORT

SPACER MESH BODY

LIGHTWEIGHT & BREATHABLE

- MATERIAL FORMS TO THE LEG
WICKS MOISTURE AWAY FROM BODY
AS WELL AS KEEPING THE LEG COOL
DURING ACTIVITIES

MALLEABLE UPRIGHTS

- ALLOWS THE UPRIGHTS TO BE BENT
TO THE CONTOUR OF THE PATIENT
FOR MAXIMUM SUPPORT AND COMFORT