

KNAPP™ II



7686 KNAPP™ II-12"
7688 KNAPP™ II-16"
HINGED KNEE ORTHOSIS

ORDERING
INFO

SIZING

MEASUREMENTS
ARE TAKEN
AT 6" ABOVE
MID-PATELLA

XS	13" - 15"	33 - 38 CM
S	15" - 18"	38 - 46 CM
M	18" - 20"	46 - 50 CM
L	20" - 23"	50 - 58 CM
XL	23" - 25"	58 - 64 CM
XXL	25" - 28"	64 - 71 CM

LARGER AND CUSTOM SIZES
FOR 7686/7688 MODELS
AVAILABLE UPON REQUEST

GREATER ROM LIMITS
AVAILABLE IN PROFESSIONAL
STOP KIT #X1224-K



DEVELOPED IN COLLABORATION WITH

THOMAS P. KNAPP, M.D.
Cedars-Sinai Kerlan-Jobe Institute



CALL TODAY! (800) 654-3241
EMAIL: ORDER@HELY-WEBER.COM



© 2019 HELY & WEBER

KNAPP™ II



THE **ULTIMATE** IN LIGHT WEIGHT
LIGAMENT SUPPORT BRACING

FEATURES & BENEFITS

ANTERIOR CLOSURE

DESIGN ALLOWS THE BRACE
TO BE CONTOURED TO GIVE
A CUSTOM FIT

EASY TO ADJUST POP STOP
HINGE ALLOWS ADJUSTMENT
IN SECONDS

COMBINATIONS OF SPACER

MESH AND DOUBLE KNIT
MATERIAL PROVIDES
COMPRESSION SUPPORT
AND EDEMA CONTROL

INDICATIONS

- MENISCUS INJURIES
- STRAINS AND SPRAINS
OF THE KNEE





MAXIMIZING JOINT SUPPORT AND PATIENT COMFORT

TWO LENGTHS:

12" PERFECT FOR THE MCL, LCL AND MENISCAL INJURIES

16" THE ADDED LENGTH AND STRAPS PROVIDE GREATER SUPPORT TO THE ACL, PCL



ANTERIOR CLOSURE
ALLOWS THE BRACE TO BE CONTOURED TO THE INDIVIDUAL'S LEG, PROVIDING A CUSTOM FIT

NEW POP STOP ROM HINGE ADJUSTS IN SECONDS (PATENT PENDING)


SLIP ROUNDED PART OF TOOL (PROVIDED) UNDER FLEXION SIDE OF CAP




REMOVE CAP WITH A GENTLY TWISTING MOTION



POP OUT EXISTING STOP WITH FORKED END OF TOOL WHILE PRESSING DOWN POST
SLIDE STOP OUT

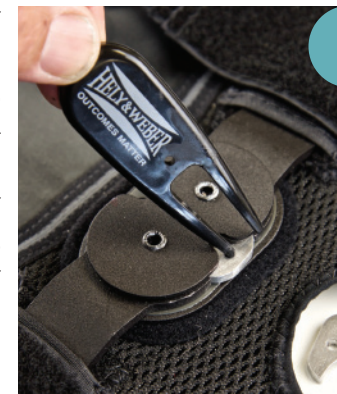

SELECT DESIRED STOP(S) AND POP STOP INTO PLACE
REPLACE CAP
NOTE: FULL INSTRUCTIONS FURNISHED WITH BRACE

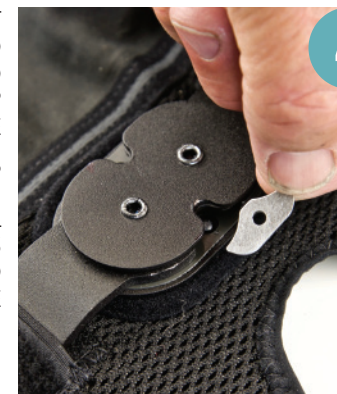

LOW PROFILE HINGE

SLEEK DESIGN

- KEEPS HINGE CLOSE TO THE KNEE FOR OPTIMUM SUPPORT

POP STOP™ ROM SYSTEM (PATENT PENDING)

- ALLOWS SETTING CHANGES IN SECONDS
- ADDITIONAL ROM STOPS INCLUDED WITH BRACE

CONDYLE PADS

- FOR ADDED COMFORT

SPACER MESH BODY

LIGHTWEIGHT & BREATHABLE

- MATERIAL FORMS TO THE LEG WICKS MOISTURE AWAY FROM BODY AS WELL AS KEEPING THE LEG COOL DURING ACTIVITIES

MALLEABLE UPRIGHTS

- ALLOWS THE UPRIGHTS TO BE BENT TO THE CONTOUR OF THE PATIENT FOR MAXIMUM SUPPORT AND COMFORT