


Tiny Titan™ Wrist Hand Thumb


455TINY

Warnings and Instructions: Review carefully, proper application is required


⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.
Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

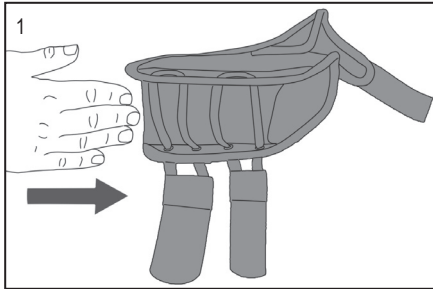


1. Slide hand into brace.

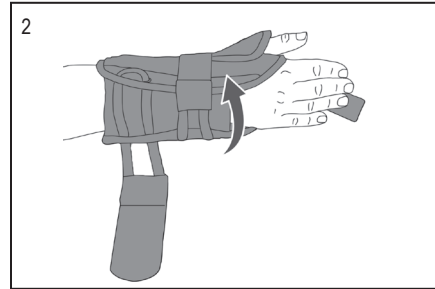


2. Pull and fasten wrist strap.
DO NOT OVERTIGHTEN

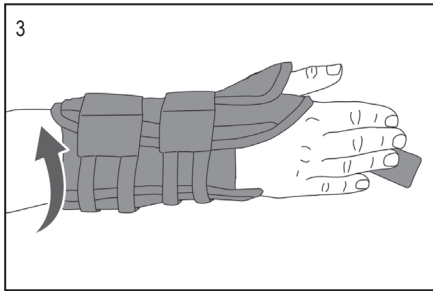

3. Pull and fasten arm strap.
DO NOT OVERTIGHTEN

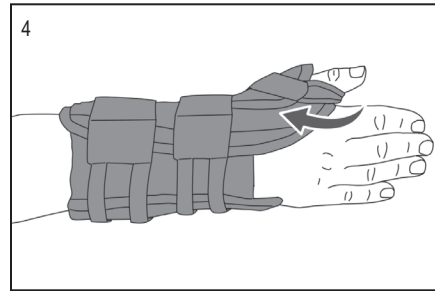

4. Pull and fasten thumb strap.
DO NOT OVERTIGHTEN

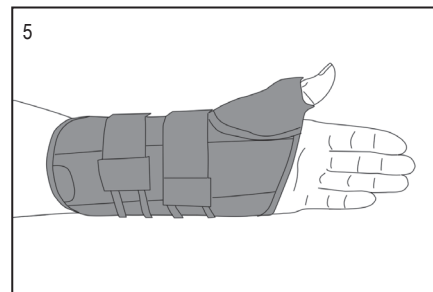

5. Completed application.
Brace should fit below midhand crease.

Application Video




Tiny Titan™ Wrist Hand Thumb


455TINY

Warnings and Instructions: Review carefully, proper application is required


⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.
Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

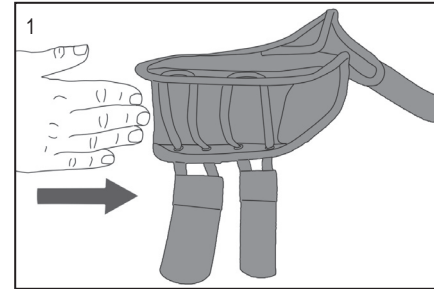


1. Slide hand into brace.

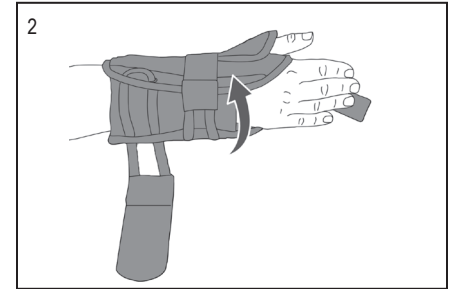


2. Pull and fasten wrist strap.
DO NOT OVERTIGHTEN

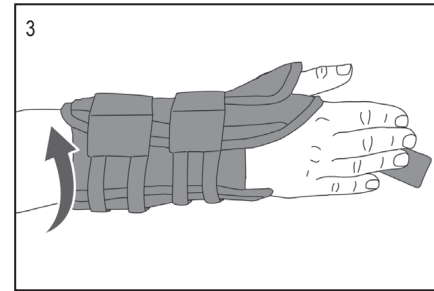

3. Pull and fasten arm strap.
DO NOT OVERTIGHTEN

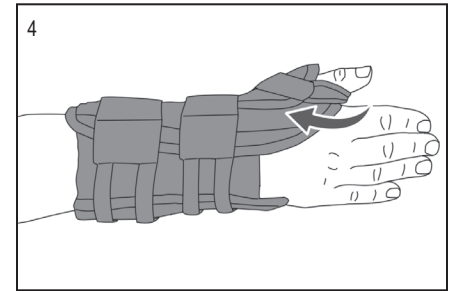

4. Pull and fasten thumb strap.
DO NOT OVERTIGHTEN

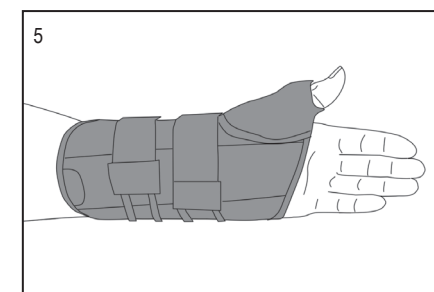

5. Completed application.
Brace should fit below midhand crease.

Application Video



© 2017 Weber Orthopedics Inc.
DBA Hely & Weber all rights reserved

Rev E 8/25



© 2017 Weber Orthopedics Inc.
DBA Hely & Weber all rights reserved

Rev E 8/25