



Warnings and Instructions: Review carefully, proper application is required

△Warning: This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.

Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

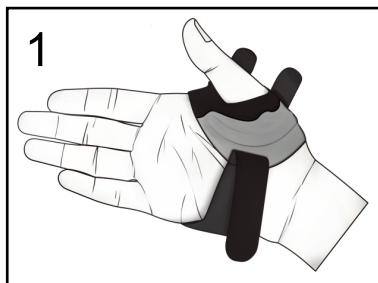


1: With all the straps loose, slide the orthosis onto the hand and over the thumb.

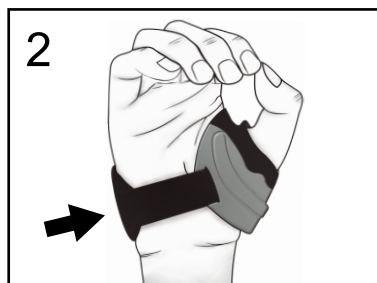

2: Position the strap guide at the ulnar border of the hand.



3: Tighten and secure the palmar strap.

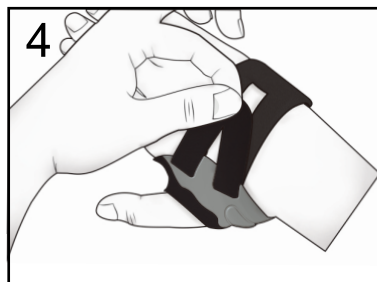

4: Tighten and secure the dorsal straps.



5: With the brace on, make an "O" with your thumb and forefinger.



6: With your thumb relaxed, squeeze the aluminum stay to custom fit the brace to your thenar contour.



Scan For
Application Video

HELY & WEBER

© 2025 Weber Orthopedics Inc
OBA Hely & Weber all rights reserved.

1185 E Main St, Santa Paula, California 93060
California: 800-221-5465, National: 800-654-3241
International: 805-525-4244, fax: 805-933-2348
U.S. fax: 800-559-5975, www.hely-weber.com

Rev A 2/25