



Warnings and Instructions: Review carefully, proper application is required

⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional. Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

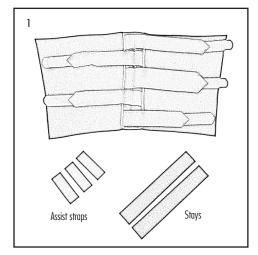


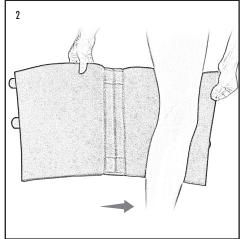

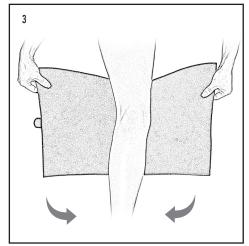


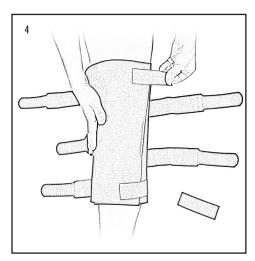


4. Use assist straps to secure the wrap

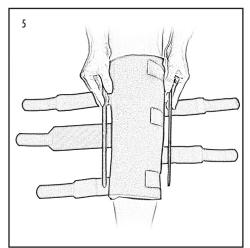


5. Place stays medial/lateral for immobilization*

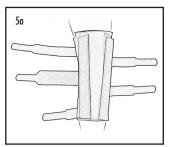

*Optional: For greater immobilization, place stays on each side of the knee cap.

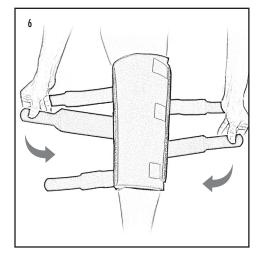

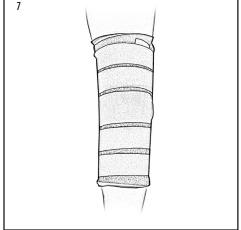



MDSS GmbH, Schiffgraben 41 30175 Hannover, Germany

DBA Hely & Weber all rights reserved 1185 E Main St., Santa Paula, California 93060 California: 800-221-5465, National: 800-654-3241 International: 805-525-4244, fax: 805-933-2348 U.S. fax: 800-559-5975, www.hely-weber.com

Rev B 2/14