


## 3 Panel Knee Immobilizer


### Warnings and Instructions: Review carefully, proper application is required

**⚠ Warning:** This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.


Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

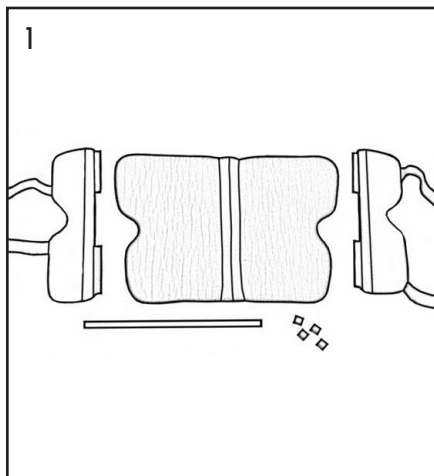


1. Separate panels, hook tabs, and patella strap

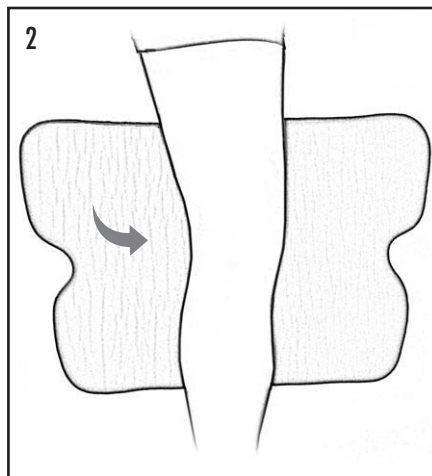


2. Center leg on curved stays along back side of brace

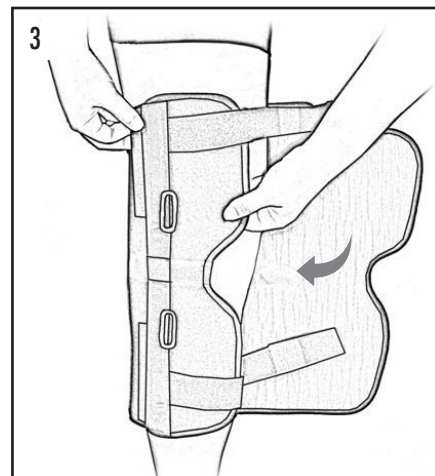


3. Position stay panel

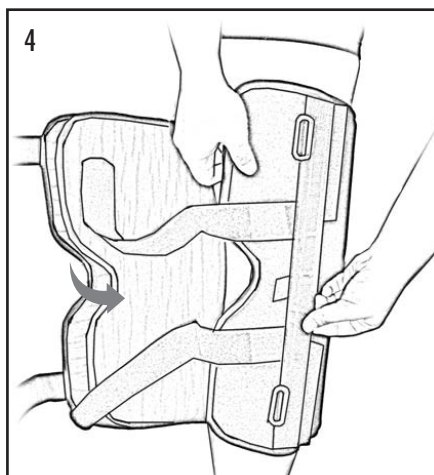


4. Position stay panel

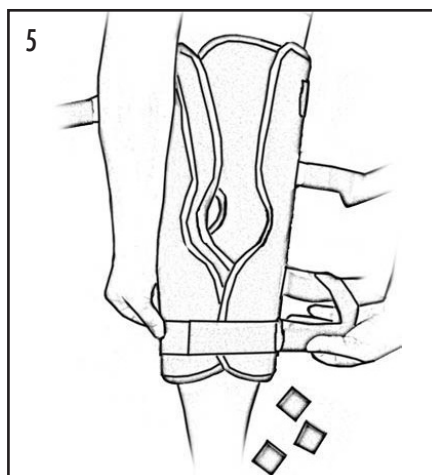


5. Pull straps through D-rings and position hook tabs for closure

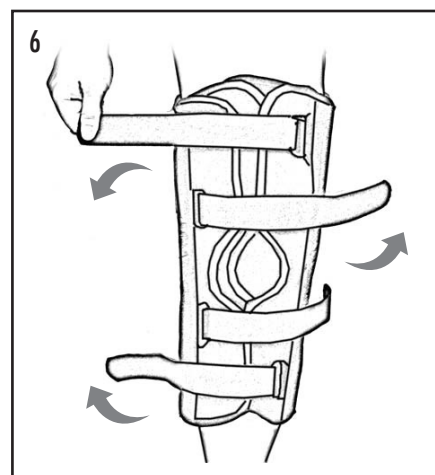

6. Secure straps

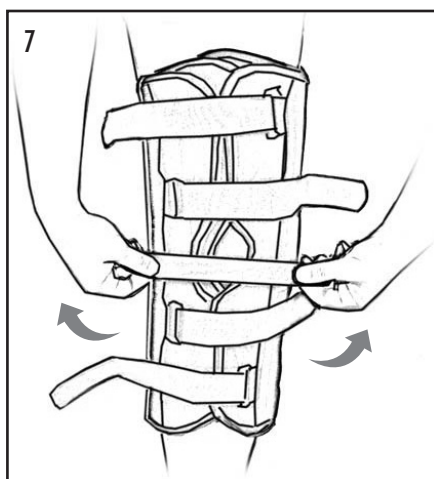

Optional 7. Apply patella strap for extra knee control

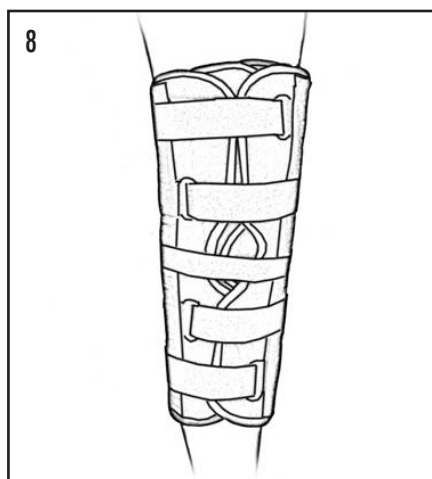

8. Trim straps. Adjust stay panels position as necessary.

**HELY & WEBER**

© 2017 Weber Orthopedics Inc.  
DBA Hely & Weber all rights reserved

1185 E Main St., Santa Paula, California 93060  
California: 800-221-5465, National: 800-654-3241  
International: 805-525-4244, fax: 805-933-2348  
U.S. fax: 800-559-5975, www.hely-weber.com

**CE**  
MDSS GmbH, Schiffgraben 41  
30175 Hannover, Germany

Rev B 2/14