
Warnings and Instructions: Review carefully, proper application is required


⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.

This device should not be worn by persons with known allergies to neoprene.

Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

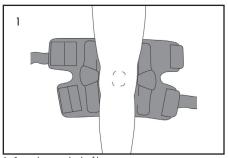

1. Center brace on back of leg.

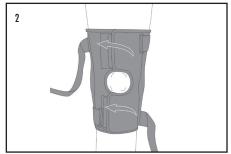


2. Pull upper and lower flaps and fasten.



3. Pull lower strap through loop lock and fasten.



Rev E 3/21

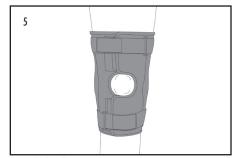

5. Finished application.


1185 E Main St., Santa Paula, California 93060 California: 800-221-5465, National: 800-654-3241 International: 805-525-4244, fax: 805-933-2348 U.S. fax: 800-559-5975, www.hely-weber.com DBA Hely & Weber all rights reserved

Knapptm Hinged Knee Orthosis 12"


Warnings and Instructions: Review carefully, proper application is required

Warning: This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional. This device should not be worn by persons with known allergies to neoprene.

Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

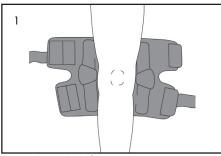

1. Center brace on back of leg.

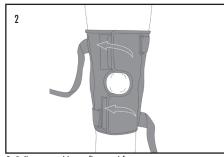


2. Pull upper and lower flaps and fasten.



3. Pull lower strap through loop lock and fasten.



4. Pull upper strap through loop lock and fasten.

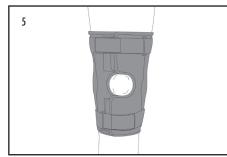

Finished application.

