Kneesio Patellofemoral Brace

Warnings and Instructions: Review carefully, proper application is required

Warning: This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional. Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry. For single patient use only.



1. Center brace over patella.



4. Undo both outer flaps. Make sure hinges are aligned with knee joint.



7. Fasten outer flap straps.





2. Pull lower strap around back of leg, through loop lock, and pull and fasten.



5. Place the buttress in the desired position.

8



8. Pull and fasten inner straps.



3

3. Pull upper straps around back of leg, through loop locks, and pull and fasten.



Hold buttress in place & pull outer flap straps to desired tension.



9. Finished application.

1185 F Main St. Santa Paula, California 93060

U.S. fax: 800-559-5975, www.hely-weber.com

California: 800-221-5465, National: 800-654-3241 International: 805-525-4244, fax: 805-933-2348 MDSS GmbH, Schiffgraben 41 30175 Hannover, Germany





7675