



Warnings and Instructions: Review carefully, proper application is required

⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional. This device should not be worn by persons with known allergies to neoprene.


Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

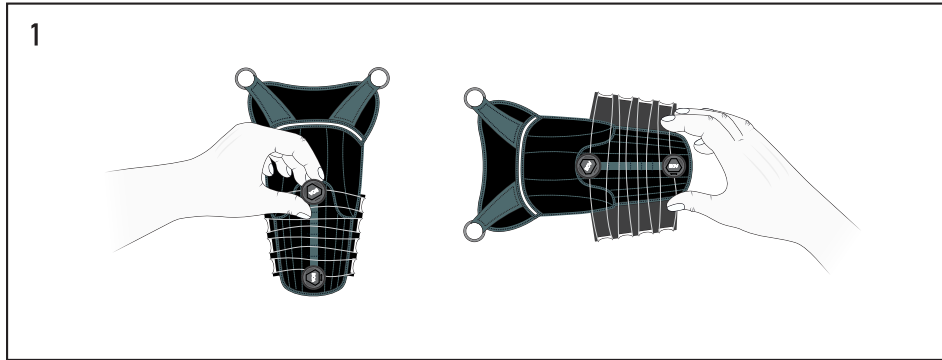


1. Prepare For Fitting. Pull up to release the BOA® Dials. Pull the laces so the brace opens up widely.

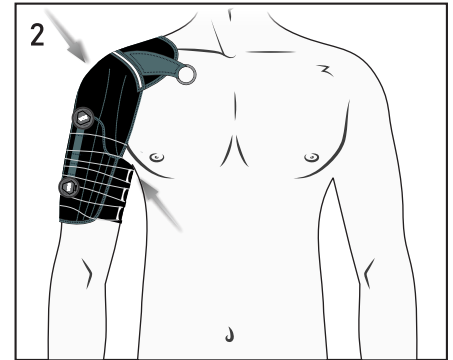


2. Slide the brace up the arm. The brace should sit at the apex of the shoulder, with the shoulder cap on top of the shoulder. The medial cuff should rest into the axilla, to avoid impingement at the elbow with flexion.

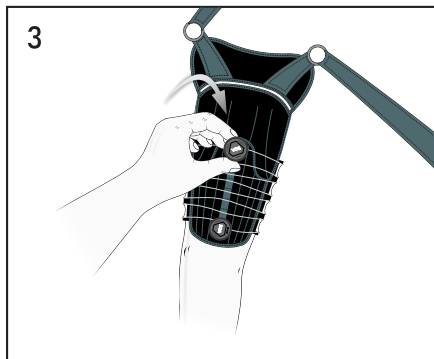


3. Gently push in each BOA® Dial to lock them. Then turn each BOA® Dial to tighten enough that the brace stays in place for the next step.

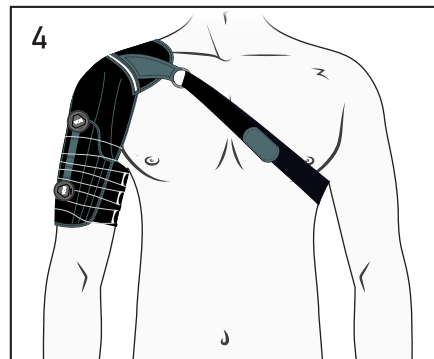


4. Bring the shoulder strap around the back, under the opposite arm and fasten. Trim the shoulder strap as needed by adjusting the alligator closure.

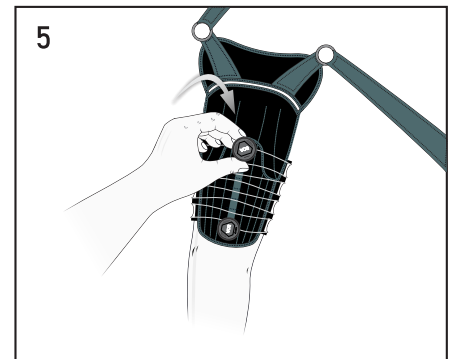

5. Make final adjustments to both BOA® Dials to achieve the desired level of compression and comfort.

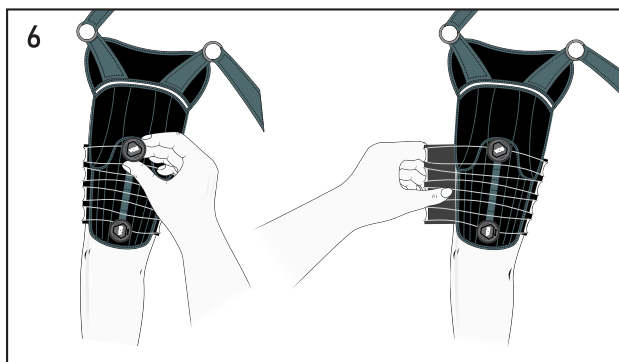

6. To remove the brace, first unfasten the shoulder strap. Next, pull up on each BOA® Dial to unlock. Then, gently loosen the brace by opening it up, and slide it down the arm.

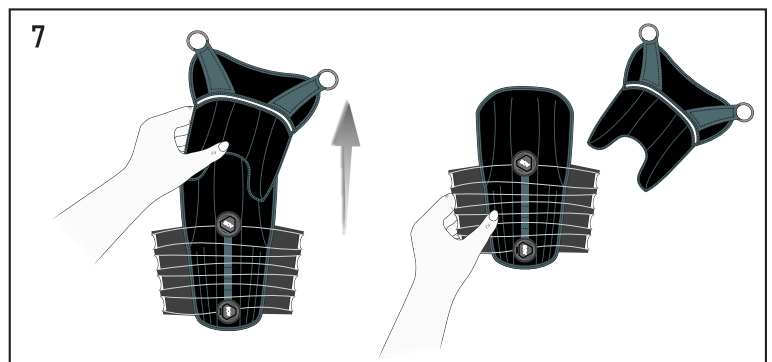

7. To use without the shoulder cap, simply remove the cap from the brace body prior to application. The aluminum stays can be bent to better conform to the shape of the shoulder.



© 2025 Weber Orthopedics Inc.
DBA Hely & Weber all rights reserved



PATENT PENDING

Application Video



Rev B 11/25