

Universal Patella Stabilizer with Stays


Warnings and Instructions: Review carefully, proper application is required

⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.


This device should not be worn by persons with known allergies to neoprene.

Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.


Left Leg Application

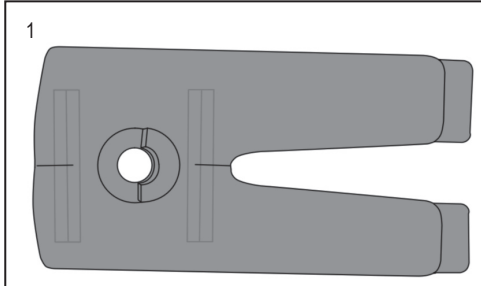


1. Place buttress on inside of brace as shown.
(Lateral placement of buttress is shown)

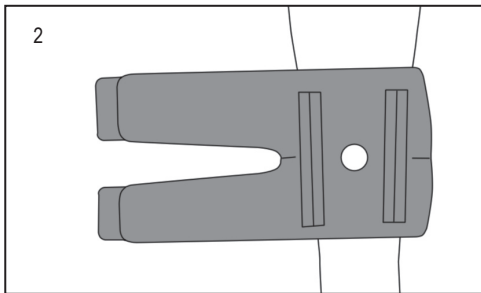

2. With the wide strap at the top, center the patella opening over the patella with the buttress against the patella.

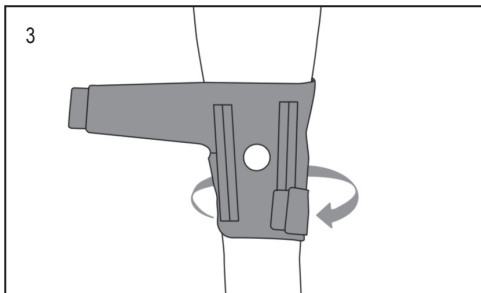


3. Pull and fasten the lower strap.

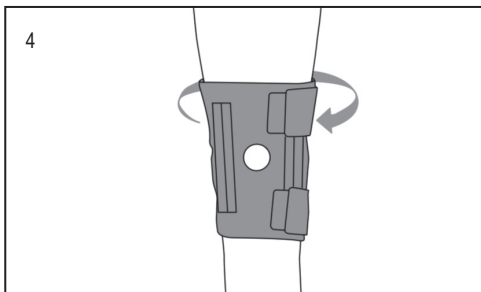


4. Pull and fasten the upper strap.


Right Leg Application

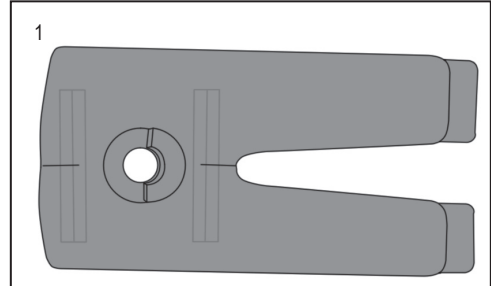


1. Place buttress on inside of brace as shown.
(Lateral placement of buttress is shown)

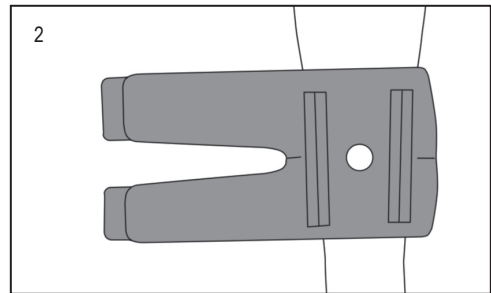

2. With the wide strap at the top, center the patella opening over the patella with the buttress against the patella.

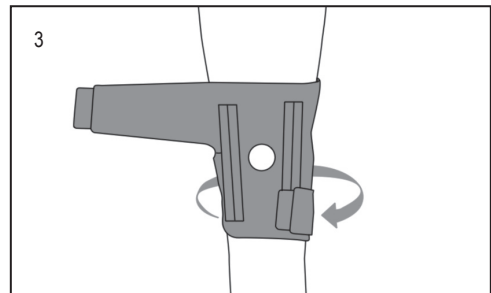

3. Pull and fasten the lower strap.

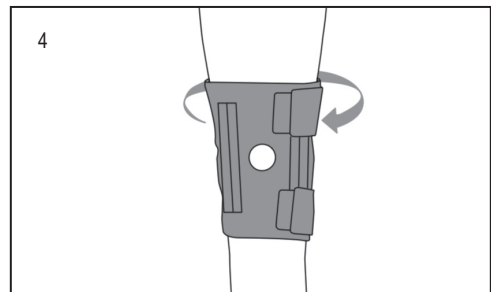

4. Pull and fasten the upper strap.

HELY & WEBER

© 2017 Weber Orthopedics Inc.
DBA Hely & Weber all rights reserved

1185 E Main St., Santa Paula, California 93060
California: 800-221-5465, National: 800-654-3241
International: 805-525-4244, fax: 805-933-2348
U.S. fax: 800-559-5975, www.hely-weber.com

CE

[EC] [REP]

MDSS GmbH, Schiffgraben 41
30175 Hannover, Germany

Rev B 9/14