


Blu Comfort Arm Sling


500B

Warnings and Instructions: Review carefully, proper application is required


⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.
Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

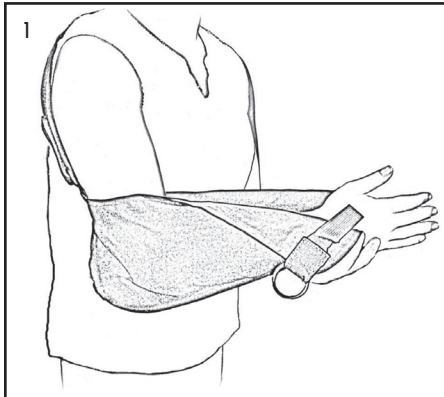


1. Place arm in the sling.

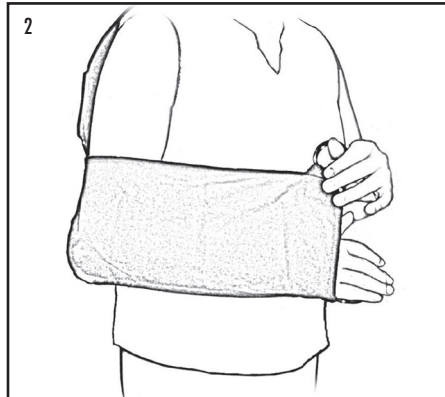


2. Pull rings together.

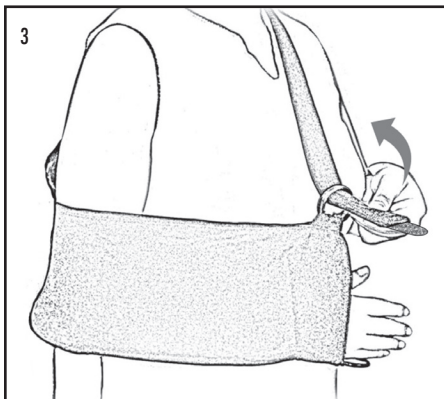

3. Pull the shoulder strap through the double rings and secure.

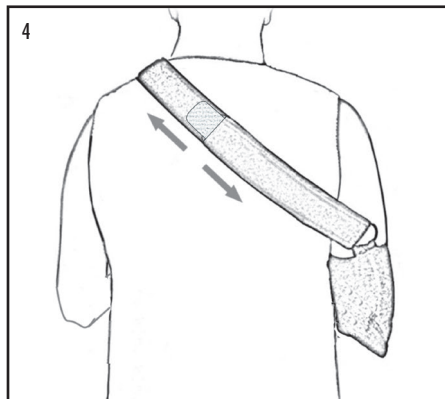

4. Adjust straps at back of brace to provide proper fit.

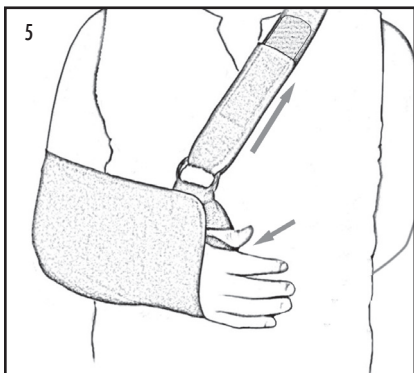

5. Adjust front strap for final fit.
Place thumb in thumb strap for comfort.

HELY & WEBER

© 2017 Weber Orthopedics Inc.
DBA Hely & Weber all rights reserved

1185 E Main St, Santa Paula, California 93060
California: 800-221-5465, National: 800-654-3241
International: 805-525-4244, fax: 805-933-2348
U.S. fax: 800-559-5975, www.hely-weber.com



MDSS GmbH, Schiffgraben 41
30175 Hannover, Germany


Rev B 8/14

Blu comfort Arm Sling


500B

Warnings and Instructions: Review carefully, proper application is required


⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.
Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

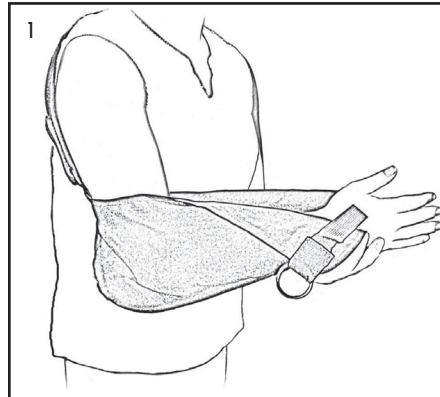


1. Place arm in the sling.

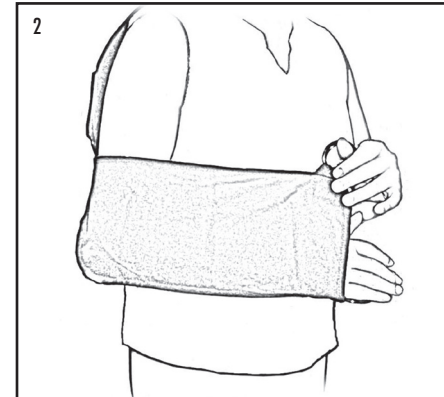


2. Pull rings together.

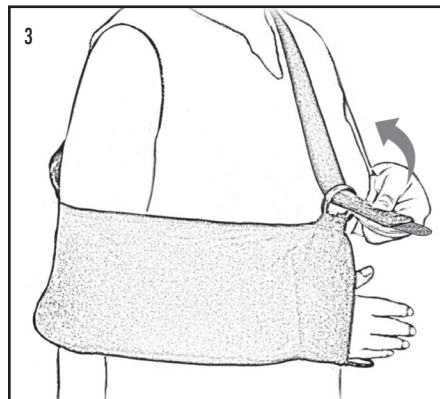

3. Pull the shoulder strap through the double rings and secure.

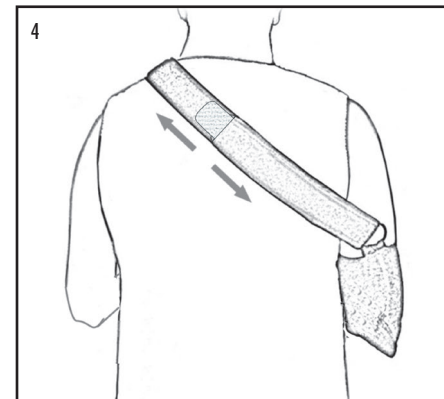

4. Adjust straps at back of brace to provide proper fit.

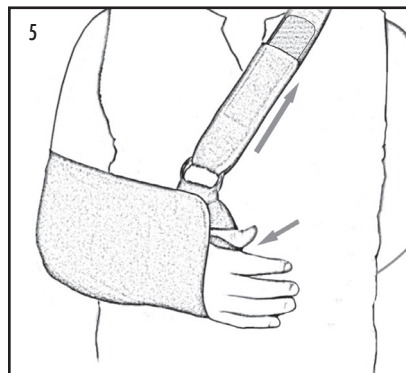

5. Adjust front strap for final fit.
Place thumb in thumb strap for comfort.

HELY & WEBER

© 2017 Weber Orthopedics Inc.
DBA Hely & Weber all rights reserved

1185 E Main St, Santa Paula, California 93060
California: 800-221-5465, National: 800-654-3241
International: 805-525-4244, fax: 805-933-2348
U.S. fax: 800-559-5975, www.hely-weber.com



MDSS GmbH, Schiffgraben 41
30175 Hannover, Germany

Rev B 8/14