


Blu Comfort Shoulder Immobilizer


Warnings and Instructions: Review carefully, proper application is required

⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.


Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

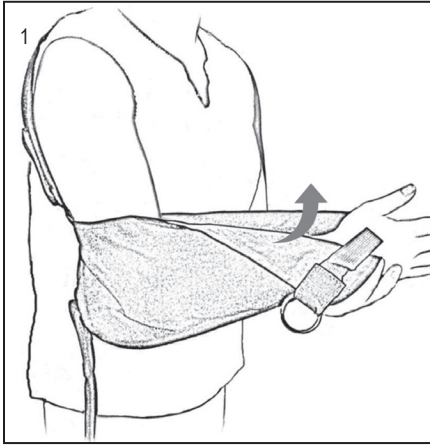


1. Place arm in sling pouch.

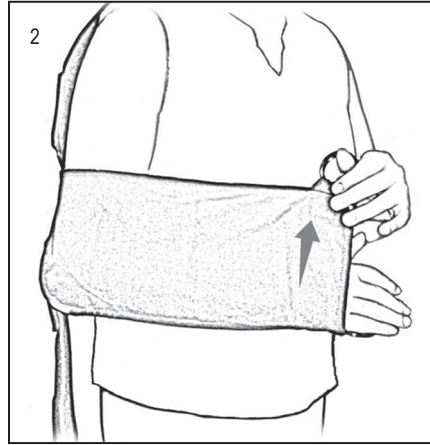


2. Pull rings together.

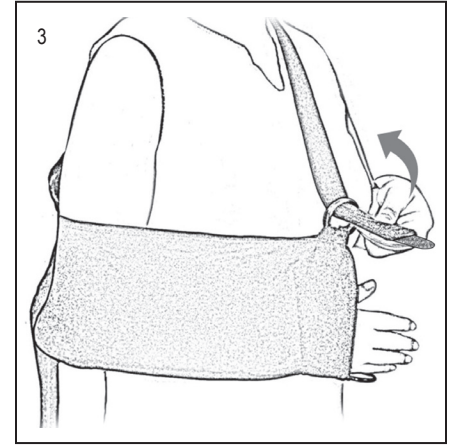


3. Pull the shoulder strap over the shoulder, through the double rings and secure.

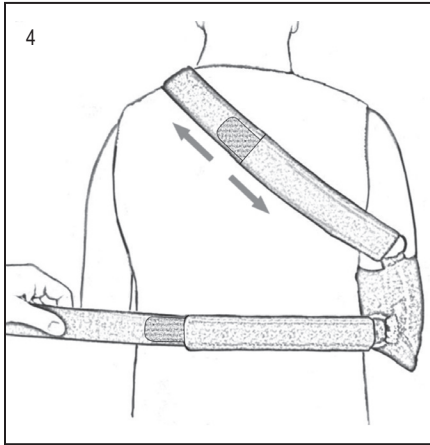


4. Locate the waist strap and bring to front.

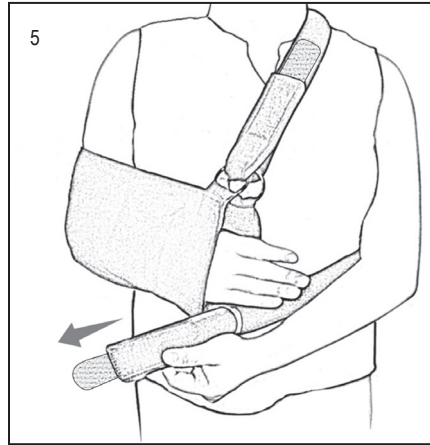


5. Pull waist strap through ring and secure.

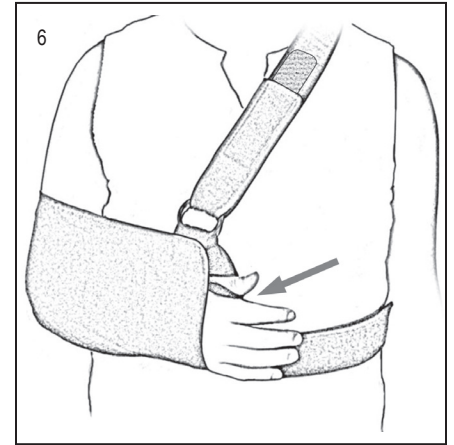

6. Place thumb in thumb strap for comfort.

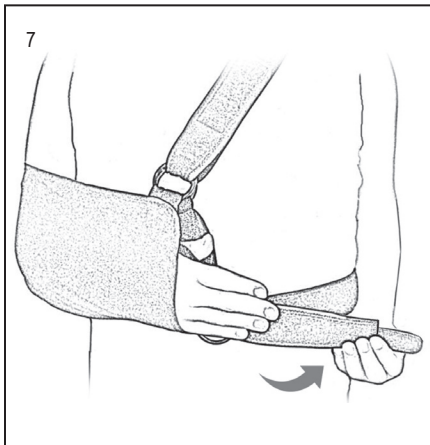

7. Adjust shoulder and waist straps as necessary to ensure a comfortable and functional fit.

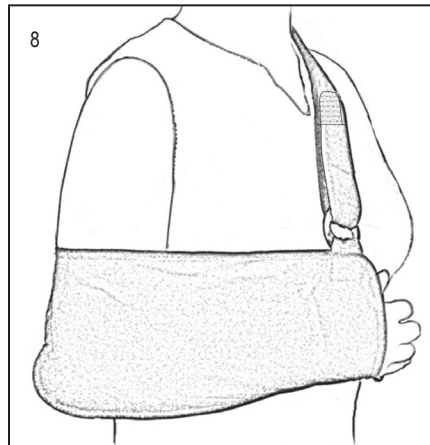

8. Finished application.