


# Knapp SAK (Shoulder Abduction Kit)


## Warnings and Instructions: Review carefully, proper application is required

**⚠ Warning:** This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.


Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

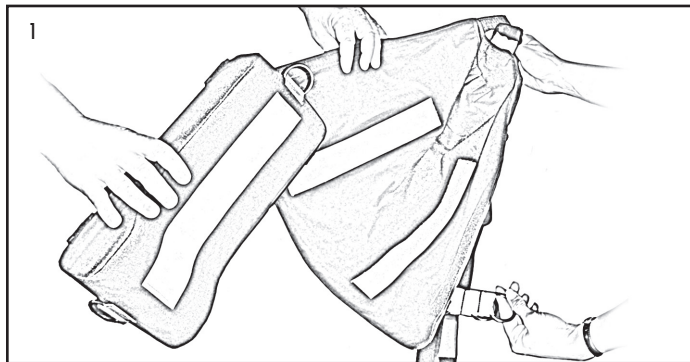


1. Position pillow inside sling for right or left arm with the wide part of pillow at sling opening. Apply blue hook to blue loop.

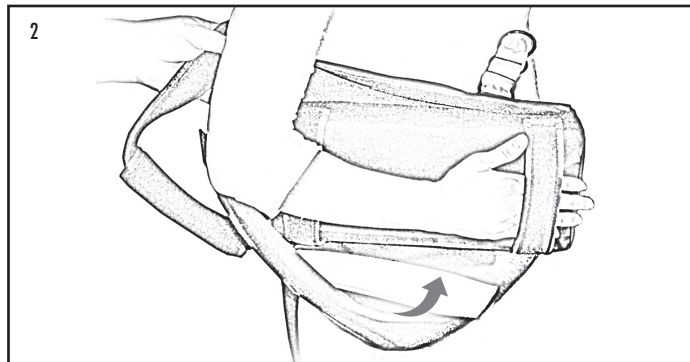


2. Place patient's arm inside sling, positioning hand through hand rest.

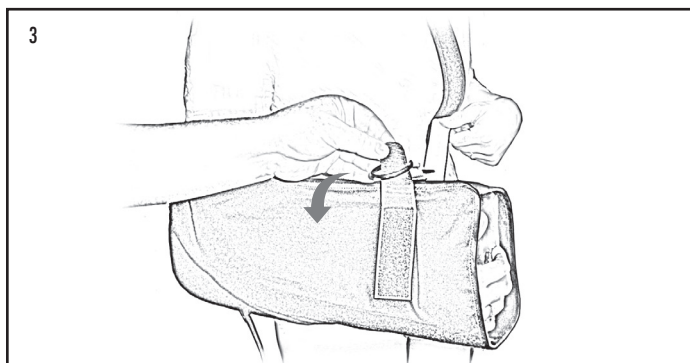


3. Enclose hand in sling, connecting yellow hook to yellow loop through the yellow loop lock on both sides of sling.

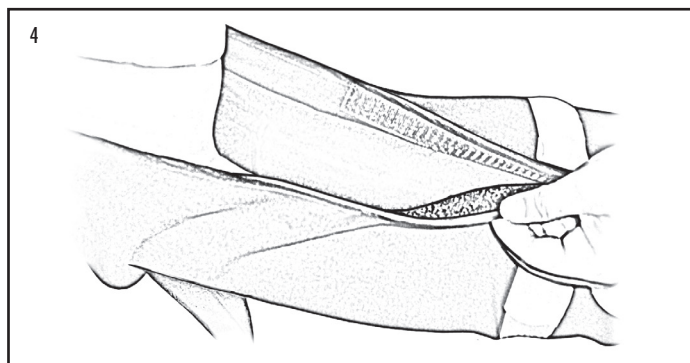


4. Secure black hook to black loop at top of sling pocket

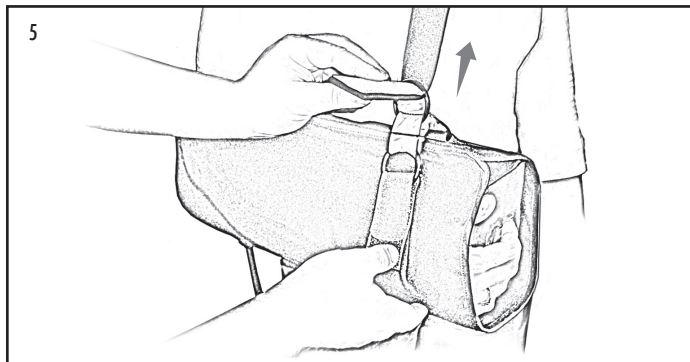

5. Apply shoulder strap, connecting white hook through white loop lock.

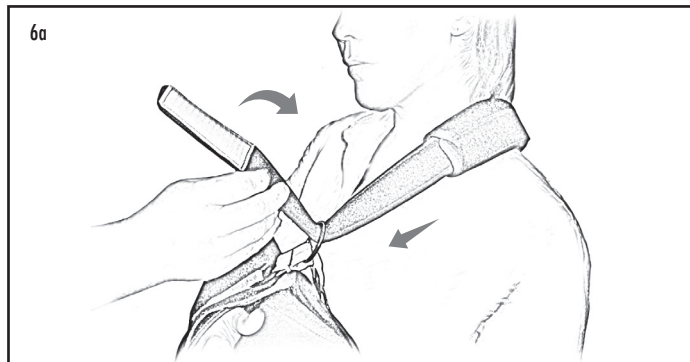

6a. Adjust both ends of shoulder strap for comfortable fit.

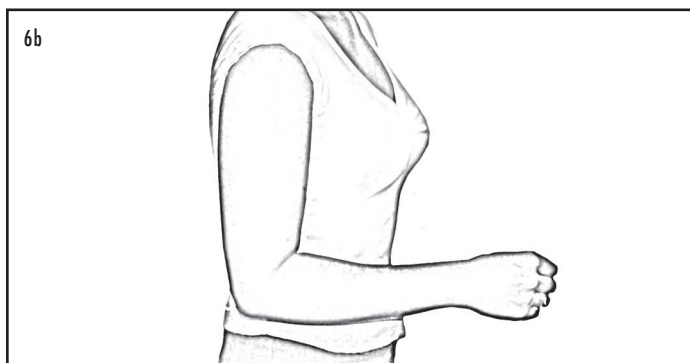

6b. Forearm and wrist should rest at 90 degrees and humerus should be in the neutral position (as shown). Adjust neck pad.

U.S. Patent No. 7189213, 7300410

**HELY & WEBER**

© 2014 Weber Orthopedics Inc.  
DBA Hely & Weber all rights reserved

1185 E Main St., Santa Paula, California 93060  
California: 800-221-5465, National: 800-654-3241  
International: 805-525-4244, fax: 805-933-2348  
U.S. fax: 800-559-5975, www.hely-weber.com



MDSS GmbH, Schiffgraben 41  
30175 Hannover, Germany


Rev B 2/14

# Knapp SAK (Shoulder Abduction Kit)


## Warnings and Instructions: Review carefully, proper application is required

**⚠ Warning:** This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.


Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

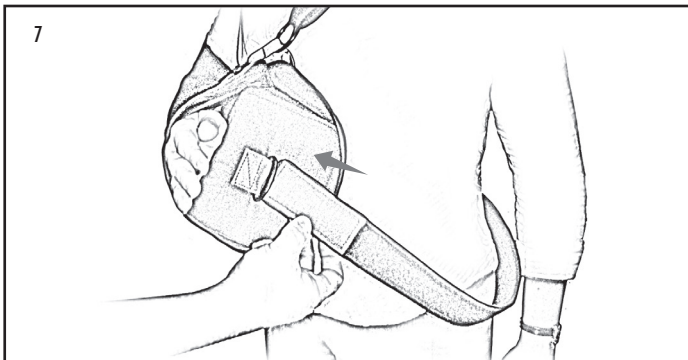


7. Apply waist strap to brace connecting red hook through red loop lock. Adjust both ends of waist strap to secure pillow and sling. This will keep the sling from rotating.

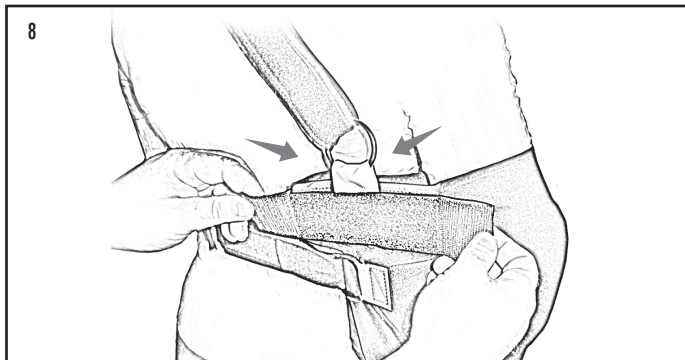


8. Tighten black hook & loop (sewn below rear white loop lock) to eliminate gapping in the elbow area.

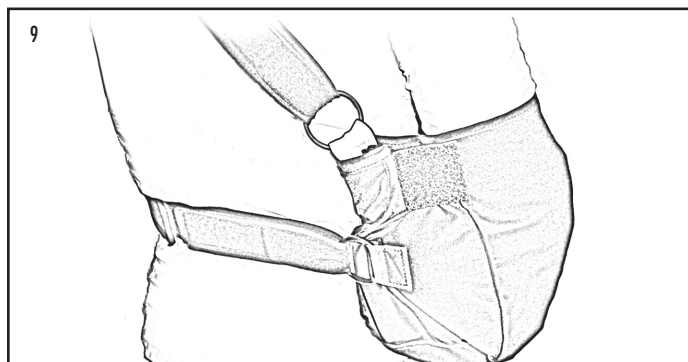

9. Rear View

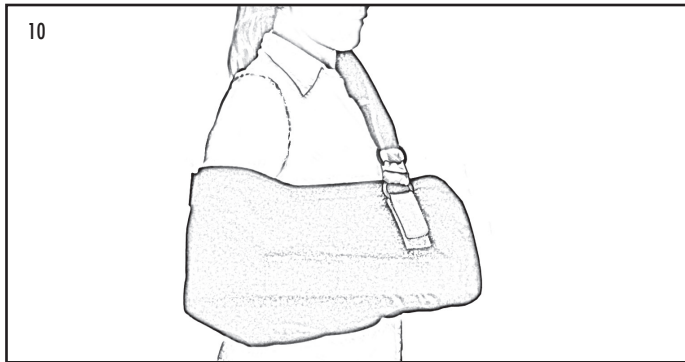

10. Side View

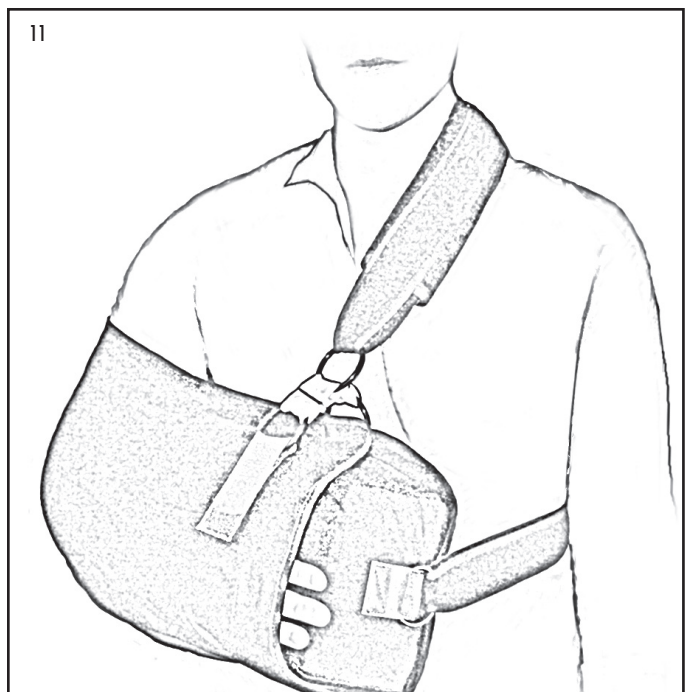

11. Front View

U.S. Patent No. 7189213, 7300410

**HELY & WEBER**

© 2014 Weber Orthopedics Inc.  
DBA Hely & Weber all rights reserved

1185 E Main St., Santa Paula, California 93060  
California: 800-221-5465, National: 800-654-3241  
International: 805-525-4244, fax: 805-933-2348  
U.S. fax: 800-559-5975, [www.hely-weber.com](http://www.hely-weber.com)

**CE**  
[EC] [REP]

MDSS GmbH, Schiffgraben 41  
30175 Hannover, Germany

Rev B 2/14