


Shoulder Houlder II Neutral


Warnings and Instructions: Review carefully, proper application is required

⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.


Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

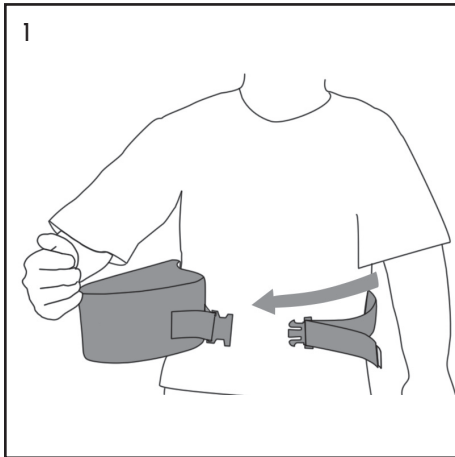


1. Place pillow against body and secure waist strap around waist.

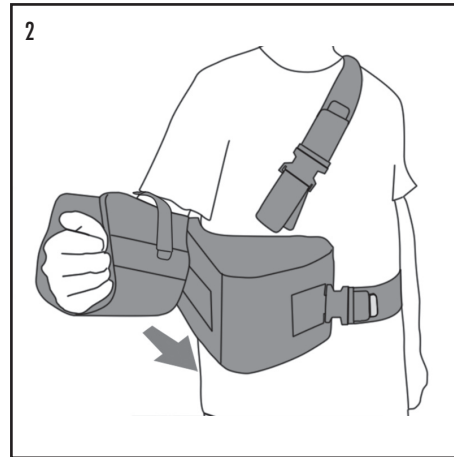


2. Place affected elbow in corner of elbow pocket. Attach brace to pillow.

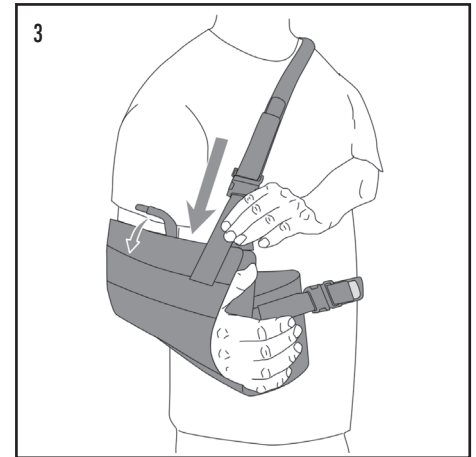


3. Fasten ends of shoulder strap to front corners of brace. Secure arm in brace with elbow strap.

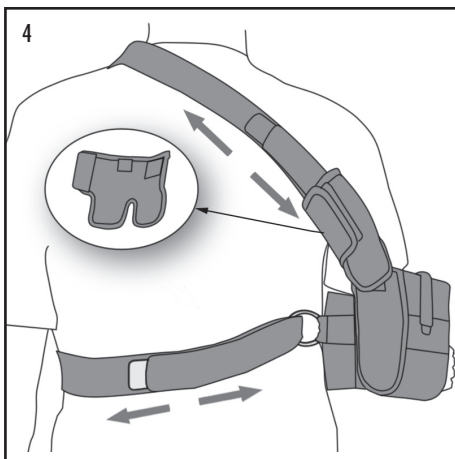


4. Have practitioner adjust back straps before first time use.
Remove comfort guard covering O ring.
Adjust straps.
Replace comfort guard.

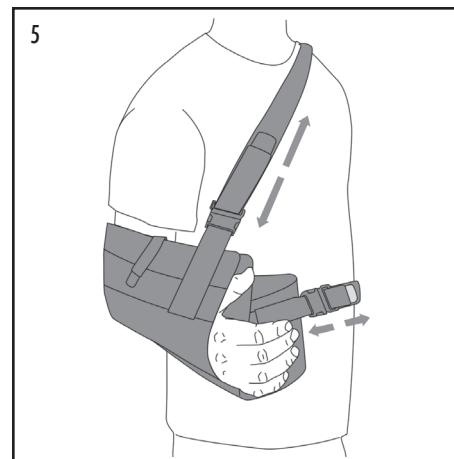

5. Make final adjustments to brace using front straps.

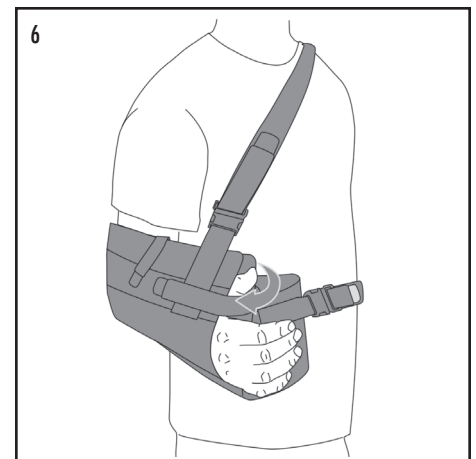

6. Secure thumb strap.

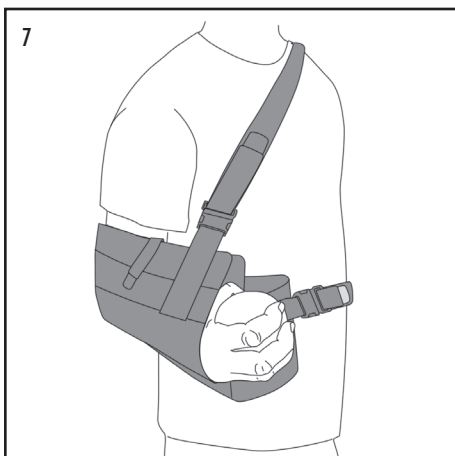

7. Completed application. Remove brace using quick release buckles.

HELY & WEBER

© 2014 Weber Orthopedics Inc.

DBA Hely & Weber all rights reserved

1185 E Main St., Santa Paula, California 93060
California: 800-221-5465, National: 800-654-3241
International: 805-525-4244, fax: 805-933-2348
U.S. fax: 800-559-5975, www.hely-weber.com

CE

EC REP

MDSS GmbH, Schiffgraben 41
30175 Hannover, Germany

Rev C 6/13