


# Universal Matt Strap

## Warnings and Instructions: Review carefully, proper application is required


**⚠ Warning:** This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.

Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.


## Sizing Customization

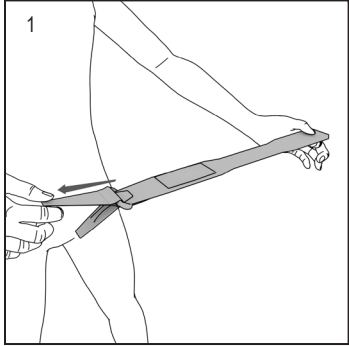


1. Detach both tabs.  
Pull tab with ring to its full extension.

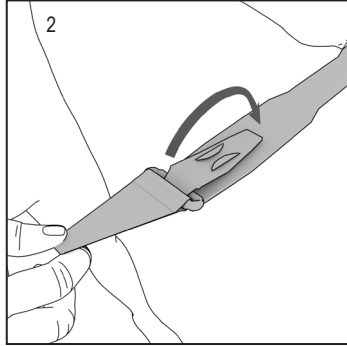


2. Fold back tab with logo and fasten.

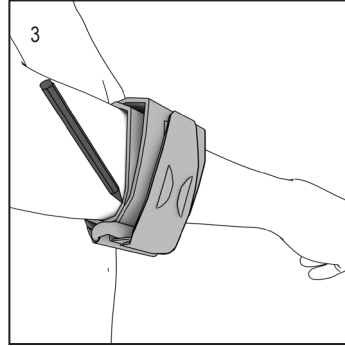

3. Fasten strap around affected area.  
Mark on logo side of ring for trimming.

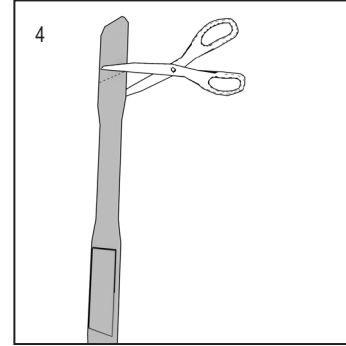


4. Remove brace and trim.

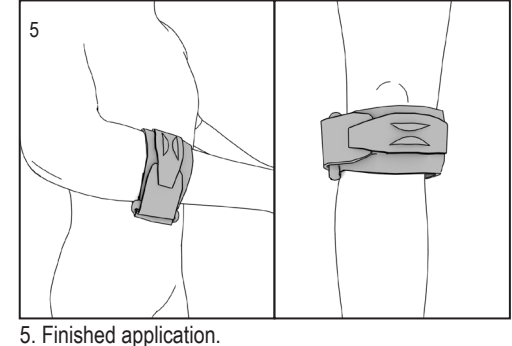


5. Finished application.


## Leg Application

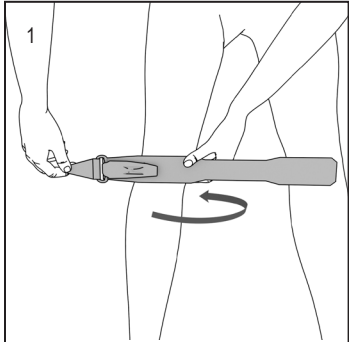


1. Place the moon shaped buttress over the affected area.  
Wrap long end of strap around leg.

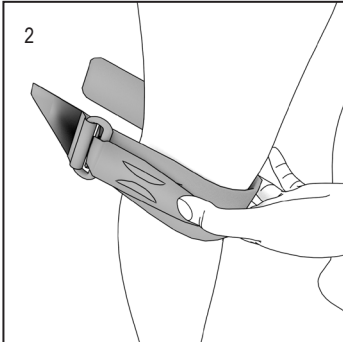


2. Mold aluminum buttress to affected area.

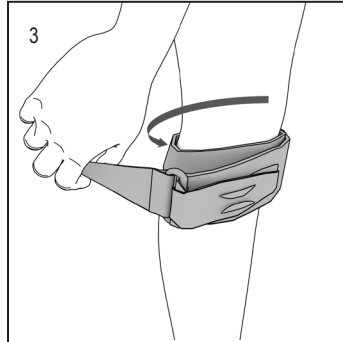

3. Hold in place and fasten strap.

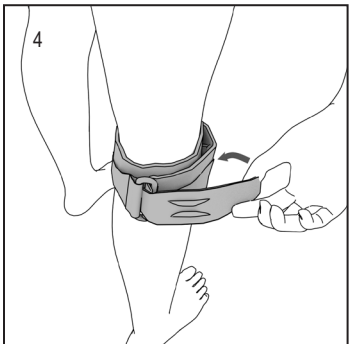

4. Detach, pull, and re-attach tab with logo.

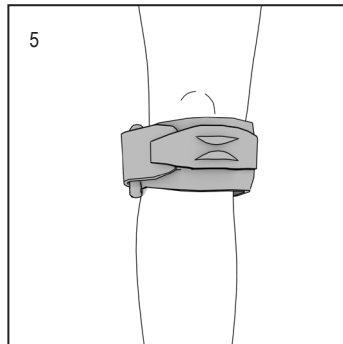

5. Finished application leg.

**HELY & WEBER**

© 2017 Weber Orthopedics Inc.  
DBA Hely & Weber all rights reserved


1185 E Main St., Santa Paula, California 93060  
California: 800-221-5465, National: 800-654-3241  
International: 805-525-4244, fax: 805-933-2348  
U.S. fax: 800-559-5975, www.hely-weber.com

**CE**  
[EC REP]


MDSS GmbH, Schiffgraben 41  
30175 Hannover, Germany

Rev B 4/16


## Arm Application

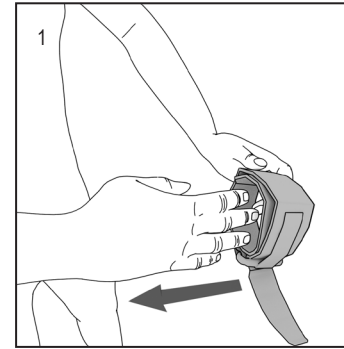


1. Adjust brace to size of affected area and fasten.  
Detach tab with logo.  
Slide brace on arm.  
Place buttress over affected area.

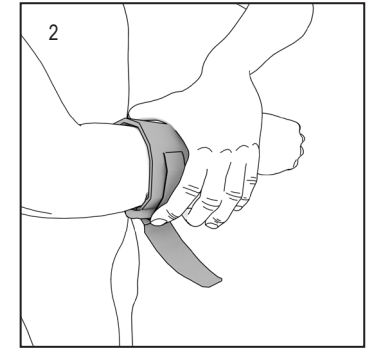

2. Mold aluminum buttress to affected area.

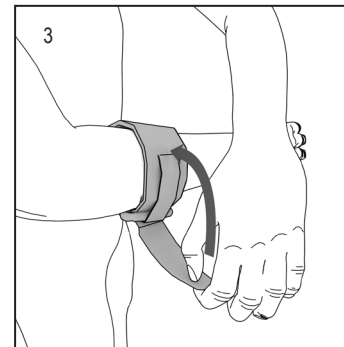

3. Pull and fasten tab with logo.

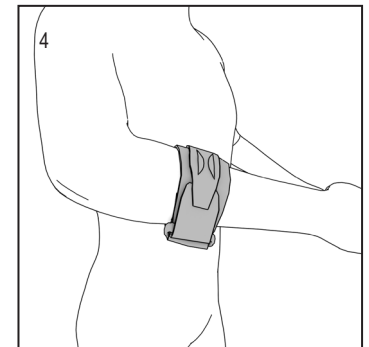

4. Finished application arm.