


Global Knee™ Adjustable Calf


Warnings and Instructions: Review carefully, proper application is required

⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.


Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

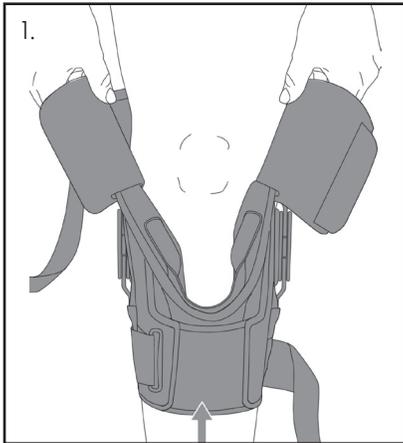


1. Loosen all straps including the inside flap, and slide brace on leg. Apply optional gel buttress to inside of buttress flap only if recommended by your health care practitioner. See buttress application instructions included with buttress.

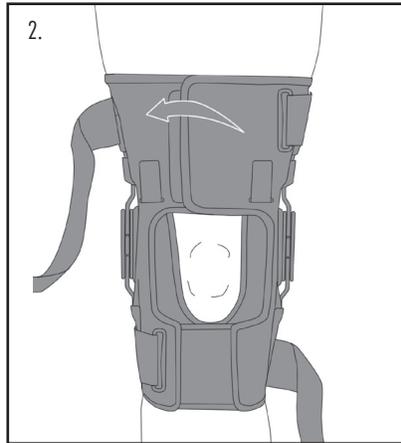


2. With leg fully extended center brace on knee. Pull and fasten upper flap. Inside buttress flap should be located just below the patella.

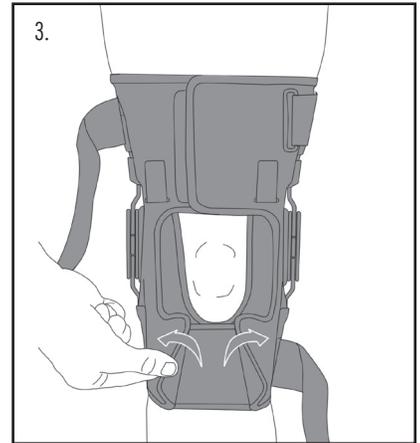


3. To adjust the calf cuff pull both flaps to disconnect the Velcro. Pull to the desired tension and reattach to the buttress flap centered on the leg.

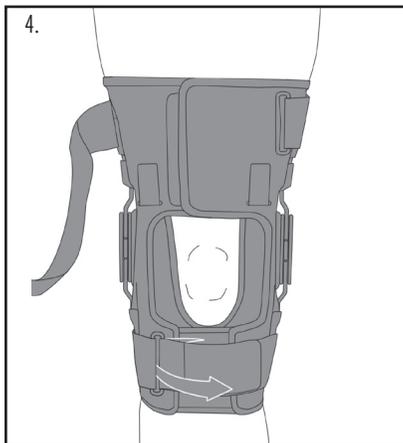


4. Pull lower strap through loop lock and fasten.

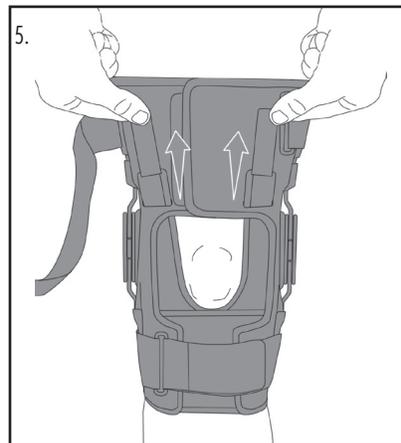

5. Pull and fasten inside buttress flap tabs as needed or instructed by your care provider.

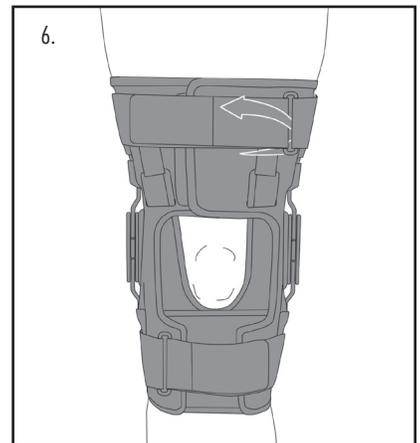

6. Pull upper strap through loop lock and fasten.

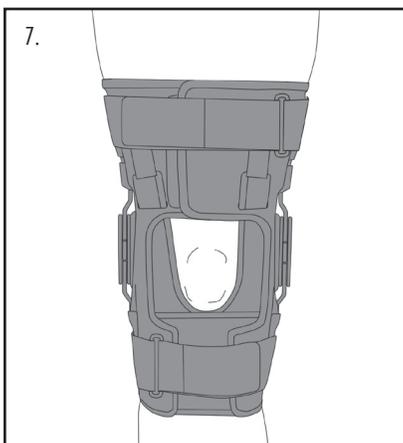

7. Completed application.

See Instructions on How to Adjust the Hinges Range of Motion on Back



Scan for video

HELY&WEBER
© 2017 Weber Orthopedics Inc.
DBA Hely & Weber all rights reserved


1185 E Main St., Santa Paula, California 93060
California: 800-221-5465, National: 800-654-3241
International: 805-525-4244, fax: 805-933-2348
U.S. fax: 800-559-5975, www.hely-weber.com

CE
[EC][REP]


MDSS GmbH, Schiffgraben 41
30175 Hannover, Germany

Rev B 10/17


Range of Motion Hinge Stop Application

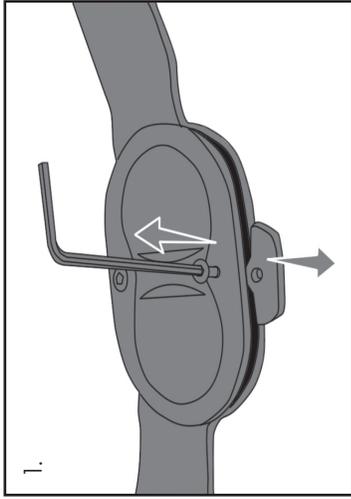


1. Use the included hex wrench to remove the existing hinge stops by removing the screws.

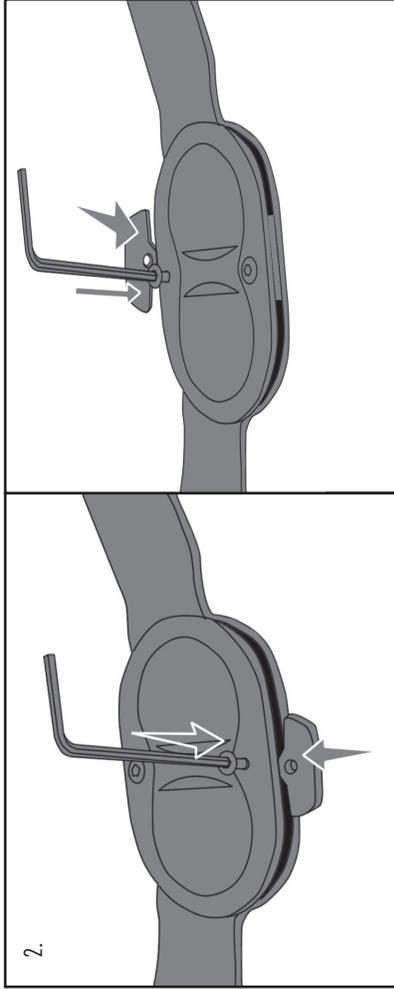

2. Choose the hinge stops with the desired degrees of extension and flexion. Install the extension stop on the extension stop side of the hinge, and the flexion stop on the flexion stop side of the hinge. Tighten the screws so they are secure but not tight. If the hinges do not move freely loosen the screws.

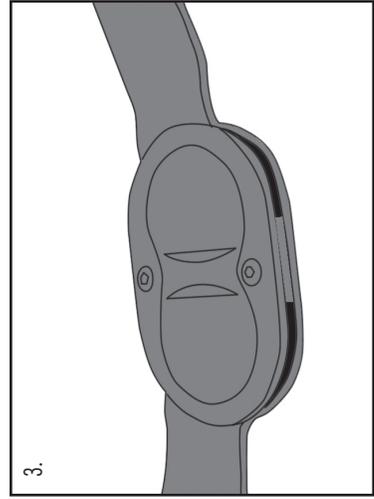


3. Completed application.
Both medial and lateral hinges must be set to the same extension and flexion.

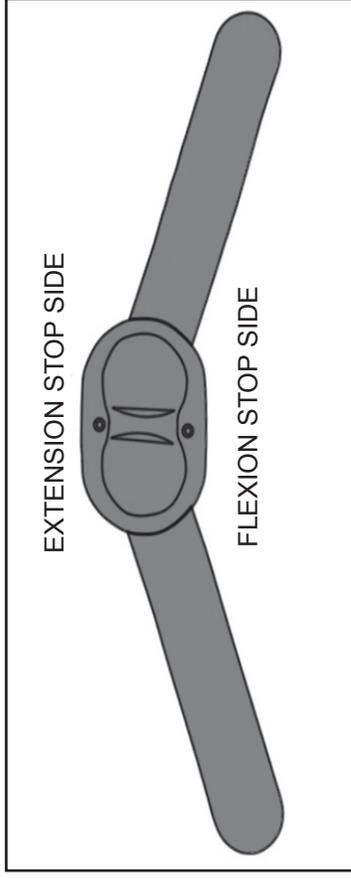

Each stop has two numbers on the front for the degree of flexion (F) or the degree of extension (E) based on which side of the hinge the stop is used.

HINGE STOP FINDER

Hinge stops shown below are included for Flexion (F) and Extension (E)
Place hinge stop on chart to verify size

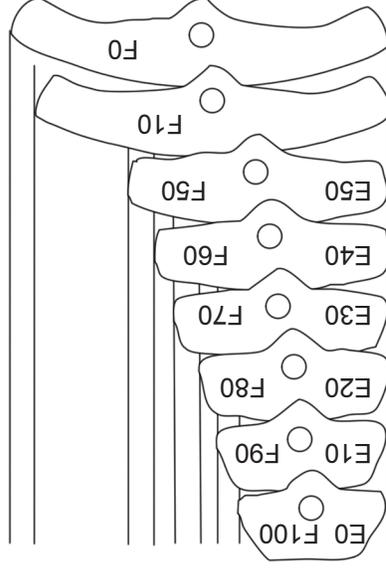

MDSS GmbH, Schiffgraben 41
30175 Hannover, Germany

1185 E Main St., Santa Paula, California 93060
California: 800-221-5465, National: 800-654-3241
International: 805-525-4244, fax: 805-533-2348
U.S. fax: 800-559-5975, www.hely-weber.com



© 2017 Weber Orthopedics Inc.
DBA Hely & Weber all rights reserved

Rev B 4/16